

COSMO*Pen*TM

The New Standard in Microneedling Technology




COSMOPen is a state-of-the-art *medical grade* high speed *Microneedling* system.

Get the same results as a fractional laser *without* pain, downtime, expense, and thermal damage.

COSMOPen can be used to treat *acne* and *surgical scars* - including *keloids* - as well as *wrinkles*, *stretch marks*, *pore size*, and signs of *aging*. Additionally, by vertically piercing the skin, **COSMOPen** helps naturally *induce collagen production* and *treat pigmentation*.


COSMOPen's new collagen creation:

- Injury
- ▼
- Stopping Bleeding
- ▼
- Inflammation
- ▼
- Growth Factors Emission
- ▼
- Fibroblast Cell Multiplication
- ▼
- Collagen And Elastin Synthesis

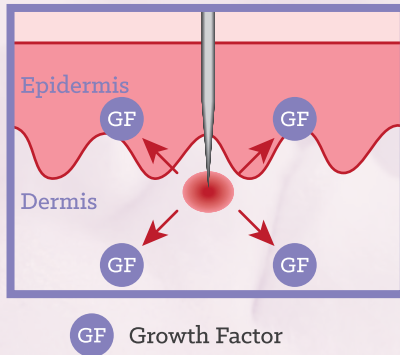

Get Visible Results!

COSMOPen uses an exclusive disposable **12-pin** needle-head, which allows the user to control the depth of penetration (**0.25 – 2.50 mm**) and speed of the needles, up to **110 cycles/second**, resulting in almost **1300 microchannels** per second, which is 20% faster than most first generation devices.

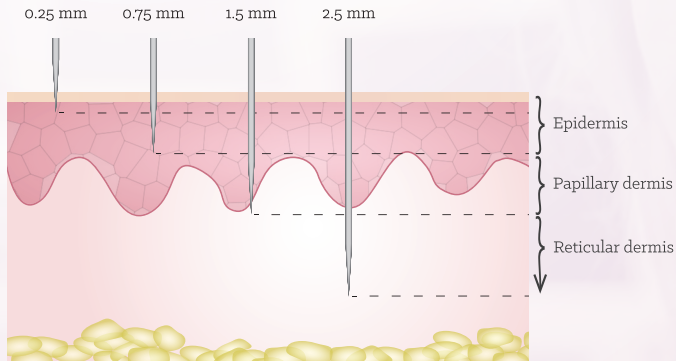

Before treatment



*After only 1 treatment
with CosmoPen & MesoMedica
MesoHairGro® Serum*




Before treatment




*After 3 treatments with CosmoPen
& MesoMedica MesoVitaTrace®
Serum*

DermaRollers

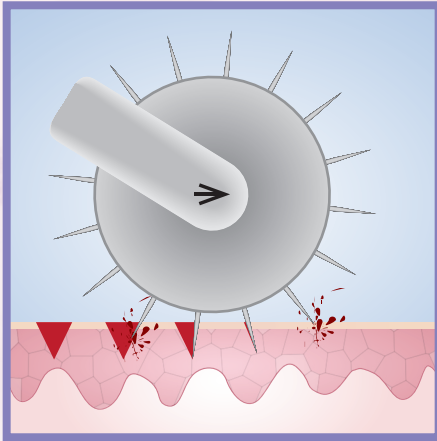

VS

COSMO*Pen*

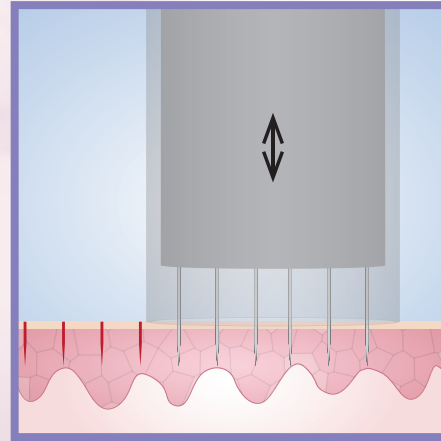

DermaRoller Flaws:

- *Excessive & undesired trauma*
- *Cross infection risks*
- *Too large for small areas*
- *Lacks accuracy & consistency for speed & depth*

COSMO*Pen* Advantages:

- *Up and down movements prevent excessive trauma*
- *Disposable pinhead - No risk of cross infection*
- *The small tip helps treat hard to reach areas*
- *Very accurate & consistent speed & depth*

Cordless VS Coded

It's all about Convenience VS *Efficiency*

When designing the CosmoPen, we chose to make a **corded** device because of the following points:

X Cordless:

Other

Pros:

Convenience
Mobility

Cons:

Inconsistent power delivery*
Power can run out during Procedure
Less powerfull
Wears out over time
Heavier – Battery weight

✓ Coded:

COSMO*Pen*

Pros:

Consistent power delivery*
Can operate all day long
Does not lose power over time – Longer life
Lighter – No battery

Cons:

Less mobility
Less convenient

***Let's get technical:**

The amount of energy available is directly related to the state of charge of the battery. A battery that is 100% charged will significantly outperform a battery that is 60% charged. It is crucial for your device to be performing consistently at the selected speed during the entire microneedling procedure.

Hyperpigmentation treatment, Melasma:



Apply **MesoClearSkin®** (*Glycolic 1% + Glutathione 2%*)

Ingredients: Aqua, Glycolic Acid, Glutathione, Sodium Chloride, Benzalkonium Chloride.

Protocol of use

Depth of application 0.25 to 1 mm depending on the skin area to be treated (see user manual)

Max dosage per session 3 to 5 ml on the face and the neck

Full treatment consists of 1 session every week for 5 to 8 weeks, then 1 maintenance session/month for 3 to 4 months.

Anti-Aging treatment, Mesolift:

Apply **MesoNutriLift®** (*Anti-Ageing Complex Mixture*)

Ingredients: Aqua, Sodium Ascorbate, Peg-35 Castor Oil, Retinyl Palmitate, Sodium Hyaluronate, Niacinamide Panthenol, Dimethyl Mea, Tocopheryl Acetate, Tiamine Nitrate, Sodium Riboflavin Phosphate, Pyridoxine Hci, Sodium Dna, Argilerine Hcl, Potassium Gluconate, Magnesium Gluconate, Sodium Gluconate, Glycine, Aspartic Acid, Proline, Hydroxyproline, Biotin, Methylsilanol Mannuronate, Sodium Benzoate, Potassium Sorbate.

Protocol of use

Depth of application 0.25 to 1 mm depending on the skin area to be treated (see user manual)

Max dosage per session 3 to 5 ml on the face and the neck

Full treatment consists of 1 session every 7 to 15 days for 5 to 8 sessions, then 1 maintenance session/month for 3 to 6 months.



Male Pattern Baldness treatment:



Apply **MesoHairGro®**

Ingredients: Panthenol, Pyridoxine, Thiamine, Biotine, Iron Gluconate.

Protocol of use

Depth of application 2.5 mm.

Max dosage per session 5 to 10ml depending of the size of the alopecia.

Full treatment consists of 1 session every week for 10 to 15 weeks, then 1 session/month for the 6 following months.

Anti-Aging & Acne treatment:

Apply **MesoVitaTrace®** (*MultiVitamins + TraceElements*)

Ingredients: Ascorbic Acid, Nicotinamide, Pyridoxine, Riboflavin Phosphate, Thiamine Hcl, Vitamin E, Vitamin A, Benzyl Alcohol, Polysorbate 80, Vitamin D, Sodium Benzoate, Potassium Sorbate, Zinc Sulfate, Copper Gluconat, Manganese Sulfate, Sodium Selenit, Chromium Chloride, Sodium Chloride, Aqua.

Protocol of use

Depth of application 0.25 to 1 mm depending on the skin area to be treated. For scars 1.25 to 1.50 mm deep
(see user manual)

Max dosage per session 3 to 5 ml on the face and the neck.

Full treatment consists of 1 session every 7 to 15 days for 6 to 8 sessions, then 1 session/ month for 3 to 6 months depending on results that have been obtained.





A PRODUCT OF

CosmoFrance_{inc}

1444 Biscayne Blvd. Suite 218
Miami, FL 33132

☎ +1 (305) 538 0110 📠 +1 (305) 675 2642



Videos & Information on:
www.cosmopen.net



Purchase supplies online:
www.cosmoFrance.net

